

A grid set for *The Grid 2* by Sensory Software International Ltd

Speedy Keys provides computer control and text communication for switch users. It has been designed to take up a small amount of space on the screen while providing all the functions you need for computer control (including keyboard, mouse and window controls).


There are also larger grids for communication with saved phrases and for creating and saving documents.

The Home page for the Speedy Keys on-screen keyboard is an alphabetically arranged keyboard, with word completion and word prediction cells, punctuation and numbers, and links to other pages. It is the start page for a series of pages associated with this User in **The Grid 2**.



Windows with a View

The Grid 2 is a Windows app that is set up to accept a number of input methods. It is easy to use a mouse, trackball, head tracking mouse, and eye gaze to operate the entire Windows computer. Or you can attach a number of different switch adapters to give you switch access with one to 12 switches to control the entire PC. Multilevel page sets with a comprehensive pre-stored vocabulary and plenty of space for personalization.

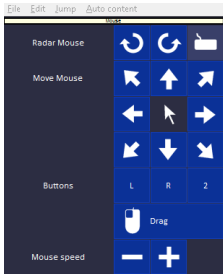

Mouse Features


Switch scanning is always challenging for PC access, but **The Grid 2** and **Speedy Keys** helps. Included in the mouse page is a Radar Mouse, latching mouse movement, mouse clicks – left, right, and double, click and drag, and the ability to control the speed of the mouse movement.

Switch Adapters

ZYGO sells the **ZyBox for Windows** USB switch adapter to allow you to connect one or two control switches to be used as input to **The Grid 2** and the Speedy Keys on-screen keyboard. We also sell the JoyCable and JoyBox adapters from Sensory Software.

Text-to-speech Chat for voice output

Integrated with the on-screen keyboard is the ability to have text-to-speech voice output. You can type letter by letter, select learning word completions and word predictions, save phrases by Topic for easy retrieval, abbreviation/expansion, and other tricks to help you speak quickly even though you are using an alternative input method.

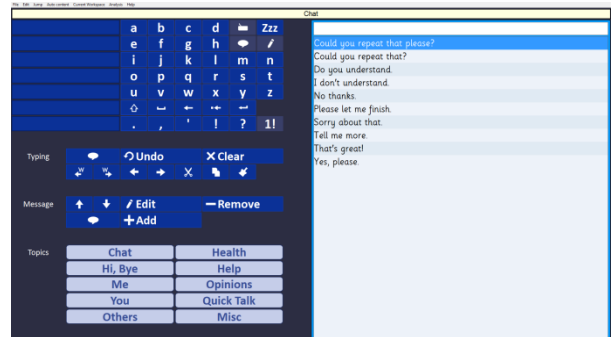

| SKU | Product | Price (USD) |
|-------------|---|--------------|
| 019-0575-04 | The Grid 2 , with Acapela, SymbolStix, Widgit Literacy Symbols | \$600.00 |
| Speedy Keys | Speedy Keys vocabulary pages for The Grid 2 | <i>free!</i> |
| 015-0300-10 | ZyBox for Windows, USB connector, one to two switch jacks | \$145.00 |
| 015-0022-00 | JoyCable USB connector, one to two switch jacks | \$150.00 |
| 015-0021-00 | JoyBox USB connector, one to 12 switch jacks | \$295.00 |