


# ZyBox 2-SC "Switch Control"



## iOS Switch Input Adapters

For iOS tablet users who cannot easily access using the touch screen, Switch Control is an accessibility feature that provides access to almost every function in iOS 7, iOS 8, and *now* iOS 9 through the use of one or multiple switches. The **ZyBox 2-SC** is the switch interface to let you take advantage of these accessibility features. Connect the **ZyBox 2-SC** to your iPad using the 30-pin or a Lightning connector, snap in one, two, or three control switches with 3.5 mm plugs, and you have access to the entire iPad.

When the **ZyBox** is plugged into your iOS device, it will power up as indicated by the LED and automatically recognize the interface. You must set up and define switch actions the first time you use Switch Control on each iOS device. But once set up you simply plug in the **ZyBox 2-SC**, plug in your switch or switches, and away you go.

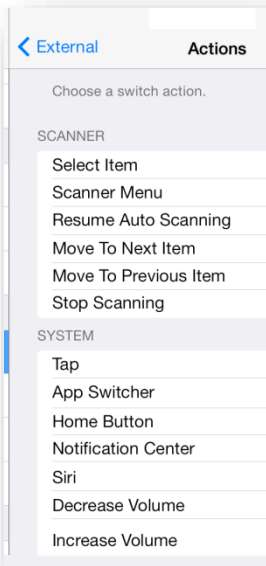


Since you plug the 30-pin and Lightning connectors to the **ZyBox 2-SC** directly to the docking connector of the iOS device, the Bluetooth connection is still available to do other things, like connect to a Bluetooth speaker, a keyboard, or to a mobile phone. It also means that the **ZyBox 2-SC** pulls very little power, unlike Bluetooth connected devices, so you will continue to enjoy long battery life. You also will not have conflicts with other Bluetooth signals.

To set up the Switch Control on your iOS device, go into Settings, move to the General tab, choose Accessibility, go into Switch Control, and turn Switch Control to the On position. You can also adjust timings and other characteristics of the scan method.

### Features of the ZyBox 2-SC (for use on iOS version 7, 8 and 9):

- 30-pin dock connector to iOS devices like the iPad 2
- Lightning to 30-pin adapter available for newer models of iOS device
- Three 3.5 mm mono jacks for external control switch inputs
- Automatic detection of switch interface when you connect it and switch(es)
- No additional software is required; set up Switch Control and switch actions first time only
- Bright LED indicator to show power On status of ZyBox for iOS
- Powered from the iOS device; no internal power required

When a user selects an item while using Switch Control, a pop-up menu appears that offers a number of advanced functions. If a user selects Scroll, Gestures, Device, or Settings from this menu, a second menu of options appears that are more advanced actions or settings for Switch Control. This provides an even greater level of control while switch scanning.

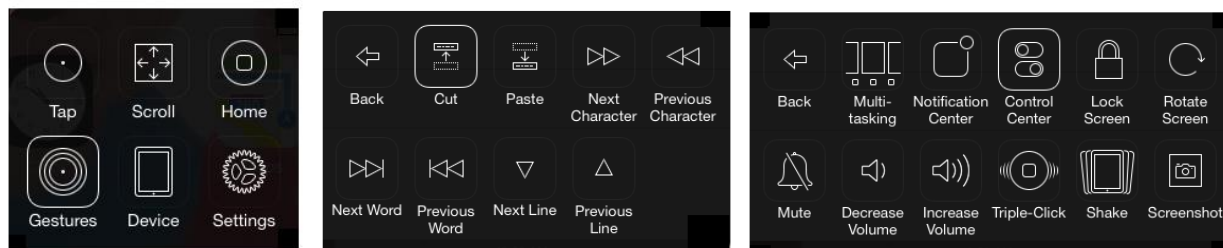

SKU	Product	Price (USD)
015-0110-1L	ZyBox 2-SC (L) "Switch Control", Lightning to 30-pin adapter, one to three switch jacks	\$195.00
015-0110-15	ZyBox 2-SC (30) "Switch Control", 30-pin connector, one to three switch jacks	\$175.00
015-0110-2L	ZyBox 2-VO (L), Lightning connector, one to three switch jacks, "VoiceOver" and key strokes	\$195.00
015-0110-25	ZyBox 2-VO (30), 30-pin connector, one to three switch jacks, "VoiceOver" and key strokes	\$175.00
015-0300-10	ZyBox for Windows, USB connector, one to two switch jacks	\$99.00